

BladeNeedleSYSTEM

Blade Remover | Needle Counter




Qlicksmart Pty Ltd.
Level 1, 148 Boundary St, West End,
Brisbane, QLD 4101, Australia
+61-7-3844-1182 T

hello@qlicksmart.com | www.qlicksmart.com

How to Use


PREPARE USING TWO HANDS:

1. Remove the sterile packaging and peel the double-sided adhesive tape. Open the BladeNeedleSYSTEM and secure the device to a surface.

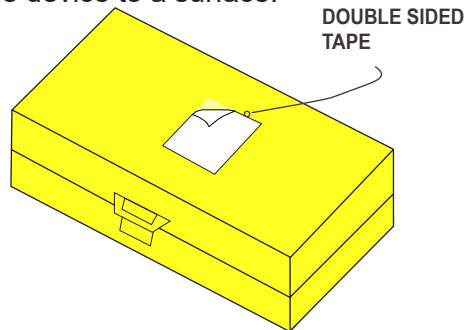

DOUBLE SIDED
TAPE


2. To remove a scalpel blade, place a cartridge into the round holder.

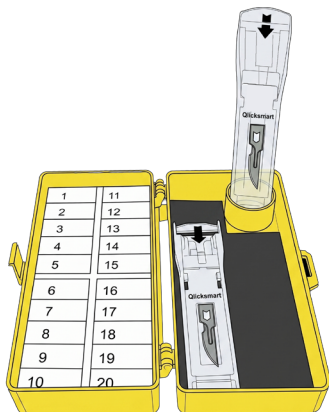


READY
FOR
USE

BLADE REMOVAL SINGLE HANDED:

3. Using one hand, slide the tip of the scalpel into the cartridge.

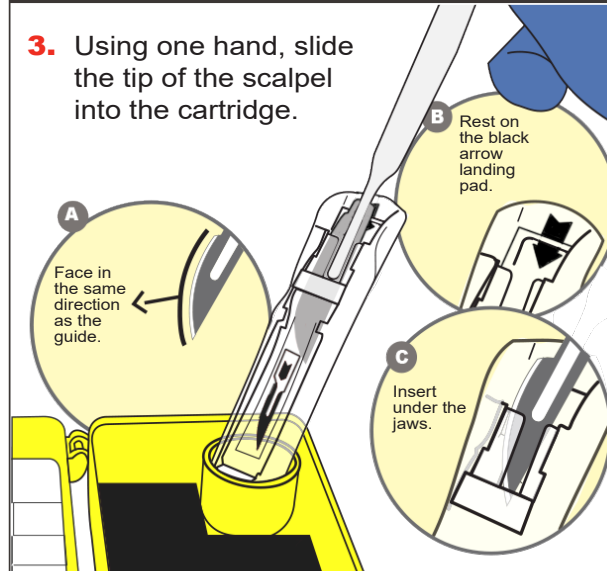


4. Tilt up to 90°, then push the scalpel down until you hear a click.

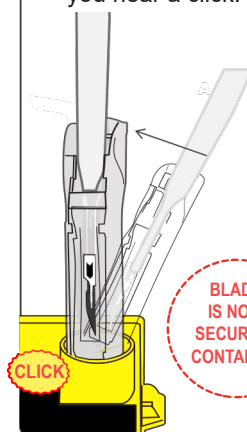


BLADE
IS NOW
SECURELY
CONTAINED

5. Remove the cartridge from the round holder.

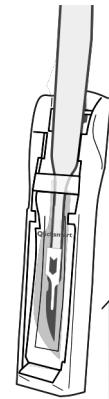


SAFE TO USE TWO HANDS

6. Using two hands, pull out the scalpel handle. The blade is now ready for counting.

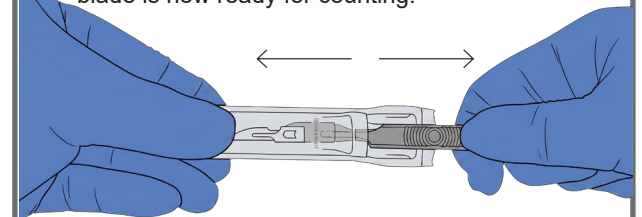


NEEDLE COUNTER

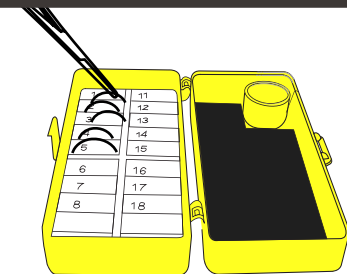

7. Insert the used needles into the foam as required. Larger needles can be placed on the flat magnetic surface.

DISPOSAL

8. Dispose of the needle counter and the cartridges separately into a sharps bin.

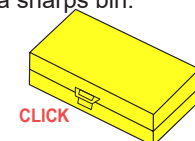

Scan the QR Code to see
the BladeNeedleSYSTEM in action.

<https://www.youtube.com/watch?v=2J49LRQ0cw4>