


100 | Scalpel Blade Removal System


## How to Use





Qlicksmart Pty Ltd. Level 1, 148 Boundary St, West End, Brisbane, QLD 4101, Australia +61-7-3844-1182 T hello@qlicksmart.com | www.qlicksmart.com

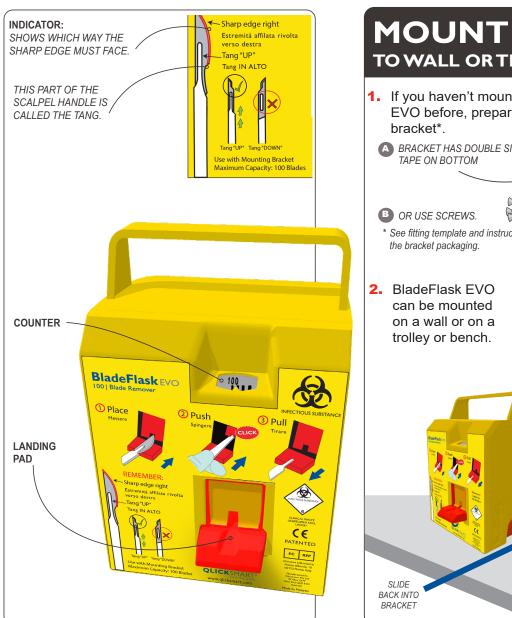

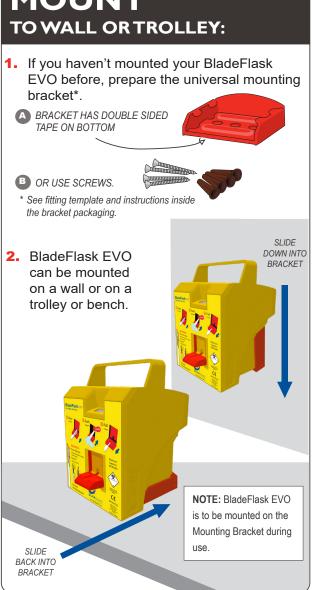

## **REMOVE BLADE** SINGLE-HANDED ACTION:

- Place blade on the landing pad with the sharp edge facing to the right (or 'tang' up).
- 2. Push up the scalpel handle into BladeFlask EVO along the ramp until you hear a click.
- 3. Remove the handle.

FULL

**NOTE:** BladeFlask EVO is disposed of as medical waste after the counter automatically shuts off when it reaches full capacity.

CLICK!