

# BladeNeedleSYSTEM

Blade Remover | Needle Counter

**Work Safe  
Today,  
Play Safe  
Tomorrow**



[www.qlicksmart.com](http://www.qlicksmart.com)

 **QLICKSMART**<sup>®</sup>  
... and be safe

# The forgotten sharps injury

Scalpel injuries are up to 6 times more likely to occur than needlestick injuries. They are particularly common in surgical settings. Regrettably, healthcare organisations and personnel have underestimated the likelihood of these injuries.

Scalpel cuts can result in infections, blood-borne diseases and psychological distress. Serious injuries could require microsurgery to repair a digital artery, nerve and/or tendon.

Many such scalpel injuries can easily be prevented using Qlicksmart's **BladeNeedleSYSTEM** (BNS), the world's pioneering single-handed blade remover.

## The world's pioneering single-handed blade remover



BNS is two safety devices in one - the world's only single-handed blade remover is combined with the standard needle counter to allow an accurate count and efficient disposal of both scalpel blades and suture needles.




BNS is simple to use with a quick "push, click, and pull" action to remove and contain used scalpel blades.



This easy-to-use solution provides a low cost alternative to safety scalpels, allowing surgeons to maintain their preferences for the standard metal handle.



Used together with a hands-free passing technique, the Qlicksmart BNS can prevent up to 5 times more injuries than a safety scalpel.

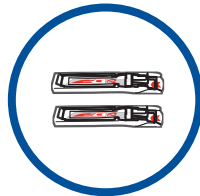

Extra blade remover cartridges are available for major surgical procedures.



It has already been embraced by healthcare workers around the world for its unique dual function, cost-saving and ease of inclusion into custom procedure packs.



The BNS contains the features of a numbered foam block for needles, a magnetic surface to hold odd-shaped sharps, and Qlicksmart's single-handed blade remover to remove used scalpel blades

# Specifications

Product Code	QBNS202S	
GMDN Code	46236	
Blade Capacity	2 cartridges in the unit - each cartridge removes 1 (one) blade.	
Compatible Blades & Handles*	Handles: #3, 3L, 4, 4L, 5, 7, 9, baron, cylindrical. Blades: #10, 10a, 11, 11P, 12, 14, 15, 15a, 15c, 16, 19, 20, 21, 22, 22a, 23, 24, 25, 25a, 27, 36, 6, 9, E11, Sabre D/15.	
Incompatible Blades & Handles	Beaver handle, Disposable handles	
Size (per unit)	Closed: Length 11.7cm Width 6.6cm Height 3.9cm	Open**: Length 11.7cm Width 12.7cm Height 8.7cm
Peso (per unità)	51gr	

Material	Needle counter box: High Impact Polystyrene Cartridges: Polypropylene (PP)
Latex	No
DEHP	No
Sterile	Yes
Sterilization Method	Gamma radiation
Re-usable	No, each cartridge removes 1 (one) blade before disposal.
Disposal method	After the blade and needle counts have been performed, the needle counter and cartridges are disposed of separately as medical wastes in a sharps container
Regulatory approvals	Australia - Listed under TGA ARTG No: 208058 USA - FDA listed Europe - Class I Sterile medical device under Council Directive 93/42/EEC and CE marked

BNSB160511A3


\* Most brands will work with Qlicksmart BladeNeedleSYSTEM.

\*\* Open box with cartridge standing in the holder


## How to Use BladeNeedleSYSTEM

### PREPARE USING TWO HANDS:

1. Peel the double sided tapes, open and stick the BNS down.

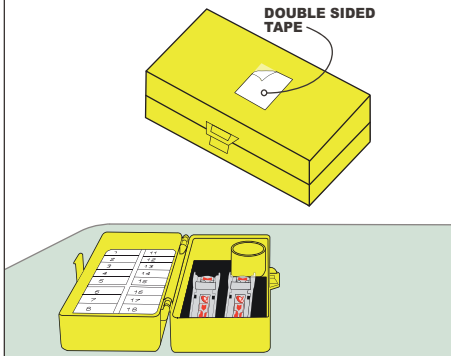


2. To remove a used scalpel blade, take a cartridge and place it inside the round holder.

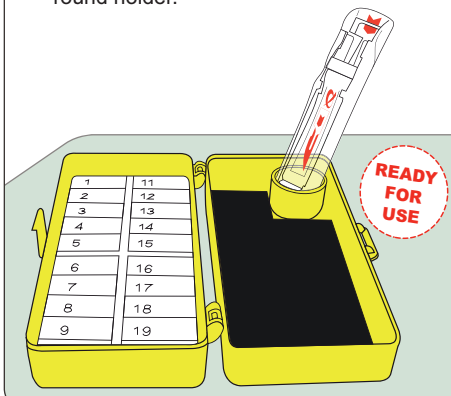


### ACTIVATE SINGLE HANDED:


3. Using one hand, slide tip of scalpel into cartridge.

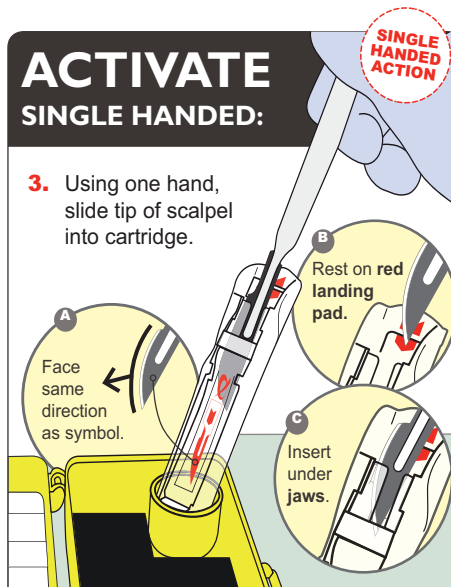

4. TILT UP to 90° then push down.

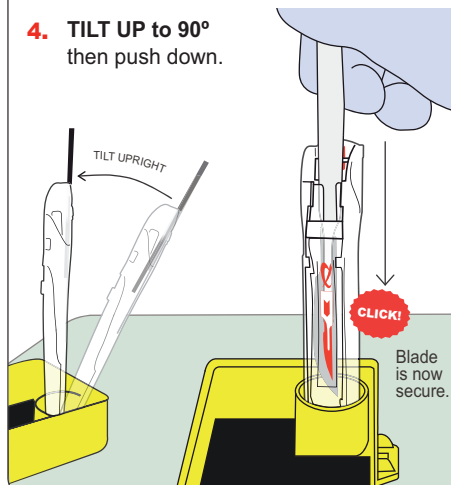


5. Remove cartridge from the round holder.

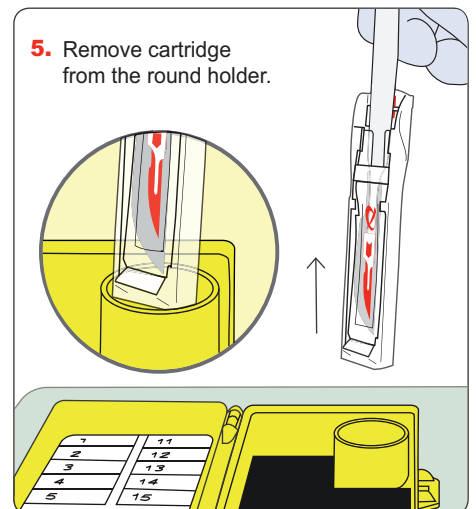


### SAFE TO USE TWO HANDS:

6. Pull out scalpel handle.

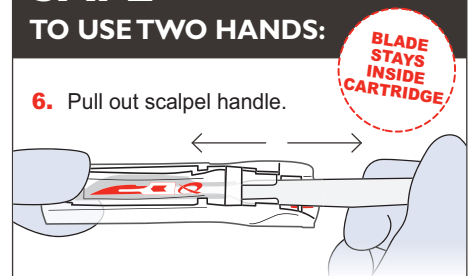

7. Ready for counting and later disposal in sharps bin.

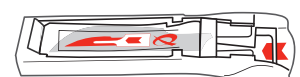


# Other Qlicksmart Products

BladeFlask EVO  
Scalpel Blade Remover




BladeFLASK Scalpel Blade Remover

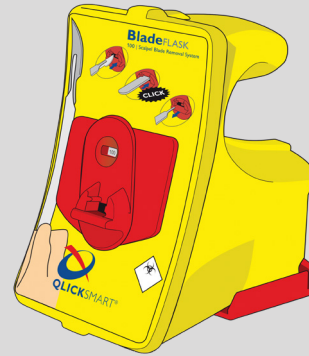

SnapIT and SnapIT Lite Ampoule Openers



BladeCASSETTE Sterile  
Scalpel Blade Remover

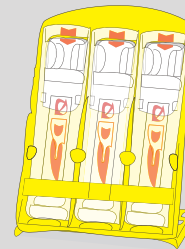

CheckCLIP Medication Identification



# About Qlicksmart

Qlicksmart Pty Ltd is committed to making healthcare professional's lives easier and safer, so that they can focus on patient care. Established in 1999 by two Australian physicians to combat scalpel cuts in hospitals, our award-winning safety devices are now being used in over 50 countries worldwide. Qlicksmart works with innovators from the healthcare sector to deliver solutions which target the overlooked concerns of nurses, surgeons, and healthcare facilities. To learn more about Qlicksmart's safety devices, please visit [www.qlicksmart.com](http://www.qlicksmart.com) or ask your distributor for a demonstration today.

## Qlicksmart Pty Ltd.



Level 1, 148 Boundary Street  
West End, Brisbane  
QLD, Australia 4101



+61 7 3844 1182



hello@qlicksmart.com  
www.qlicksmart.com



Search "qlicksmart"  
and stay connected



Scan the QR Code  
to learn more  
about the BNS.



Distributed by: