

# BladeSINGLE

One | Blade Remover

**Work Safe  
Today,  
Play Safe  
Tomorrow**



[www.qlicksmart.com](http://www.qlicksmart.com)

 **QLICKSMART®**  
... and be safe


# The forgotten sharps injury

Scalpel injuries are up to 6 times more likely to occur than needlestick injuries. They are particularly common in surgical settings. Regrettably, healthcare organisations and personnel have underestimated the likelihood of these injuries.

Scalpel cuts can result in infections, blood-borne diseases and psychological distress. Serious injuries could require microsurgery to repair a digital artery, nerve and/or tendon.

Many such scalpel injuries can easily be prevented using Qlicksmart's **BladeSINGLE**, the world's pioneering single-handed blade remover.

## The world's pioneering single-handed blade remover

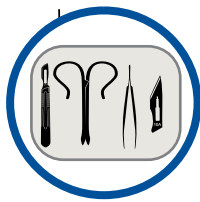

The BladeSINGLE is perfect for smaller surgical procedures, removes a single surgical blade for an accurate count and efficient disposal.



The BladeSINGLE is simple to use with a quick "push, click, and pull" action to remove and contain used scalpel blades.



This easy-to-use solution provides a lower cost alternative to safety scalpels.



Used together with a hands-free passing technique, the BladeSINGLE can prevent up to 5 times more injuries than a safety scalpel.



The BladeSINGLE allows surgeons to maintain their preferences for the standard metal handles.



It has already been embraced by healthcare workers around the world for its unique dual function, cost-saving and ease of inclusion into custom procedure packs.

Workplace safety is the right of all healthcare workers, and the responsibility of all employers.

Use the Qlicksmart BladeSINGLE and be safe.



# Specifications

Product Code	QSBS01	
GMDN Code	46236	
Blade Capacity	1 blade - Each BladeSINGLE cartridge removes 1 blade.	
Compatible Blades & Handles*	Handles: #3, 3L, 4, 4L, 5, 7, 9, Baron, Lawton long handle, cylindrical handle Blades: #10, 10a, 11, 11P, 12, 14, 15, 15a, 15c, 16, 19, 20, 21, 22, 22a, 23, 24, 25, 25a, 27, 36, 6, 9, E11, Sabre D/15.	
Incompatible Blades & Handles	Beaver handle, Disposable handles	
Size (per unit)	Folded: Length 9.3cm Width 2.2cm Thickness 0.9cm	Standing: Length 7.3cm Width 2.2cm Height 9.3cm
Weight (per unit)	< 10g	

Material	Polypropylene (PP)
Latex	No
DEHP	No
Sterile	Yes
Sterilization Method	Gamma radiation
Re-usable	No.
Disposal method	After the blade count has been performed, the whole BladeSINGLE is disposed of as medical waste in a sharps container.
Regulatory Approvals	Independently tested by a NATA accredited laboratory for compliance with AS 3825:2020 Standard (procedures and devices for the removal, containment and disposal of scalpel blades from scalpel handles). Australia - Listed under TGA ARTG No: 134623 USA - FDA listed Europe - Class I Sterile medical device under Council Directive 93/42/EEC and CE marked

BSBI60510A4-US


\* Most brands work with Qlicksmart BladeSINGLE

## How to Use BladeSINGLE


### PREPARE

USING TWO HANDS:

1. Peel to reveal adhesive.

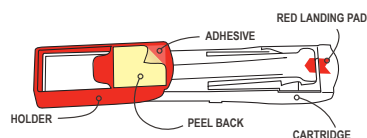

2. Take cartridge out of holder.

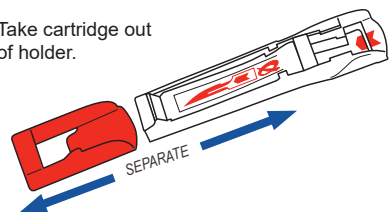

3. Fasten holder to drape or flat surface.




4. Stand in holder.




### ACTIVATE

SINGLE-HANDED ACTION:


5. Using one hand, slide scalpel tip into the cartridge.

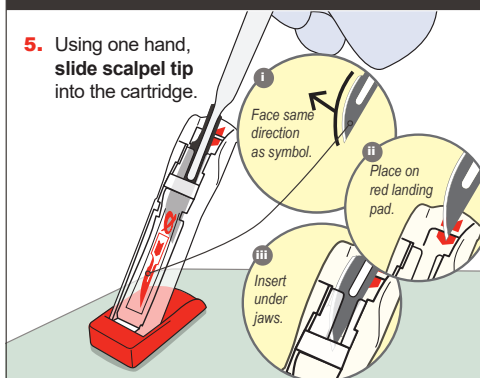

6. Tilt BladeSINGLE up then push down until you hear a click.

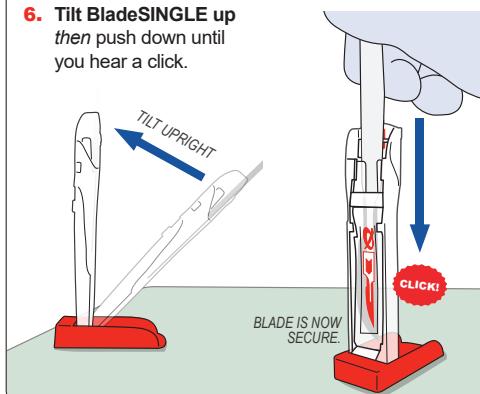


7. Remove cartridge from holder.

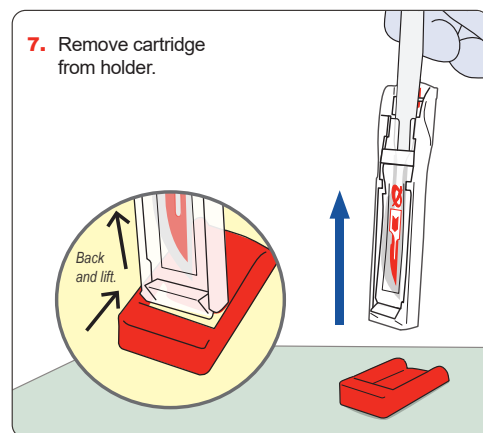

### SAFE

TO USE TWO HANDS:

8. Pull out scalpel handle.

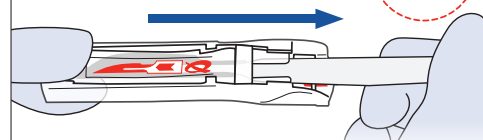


9. Ready for counting and later disposal in sharps bin.



## Other Qlicksmart Products

BladeFLASK Scalpel  
Blade Remover

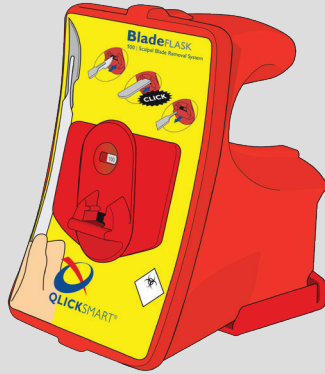


BladeCASSETTE Sterile  
Scalpel Blade Remover



SnapIT and SnapIT Lite Ampoule Openers



BladeNeedleSYSTEM Sterile 2-in-1  
Needle Counter + Blade Remover

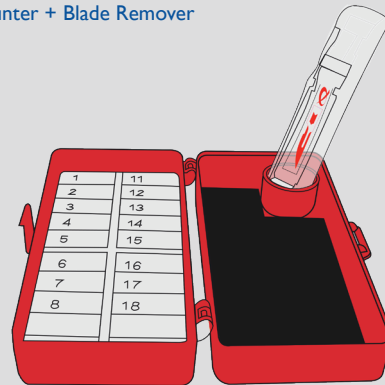

CheckCLIP Medication Identification



## About Qlicksmart

Qlicksmart Pty Ltd is committed to making healthcare professional's lives easier and safer, so that they can focus on patient care. Established in 1999 by two Australian physicians to combat scalpel cuts in hospitals, our award-winning safety devices are now being used in over 50 countries worldwide. Qlicksmart works with innovators from the healthcare sector to deliver solutions which target the overlooked concerns of nurses, surgeons, and healthcare facilities. To learn more about Qlicksmart's safety devices, please visit [www.qlicksmart.com](http://www.qlicksmart.com) or ask your distributor for a demonstration today.

### Qlicksmart Pty Ltd.



Level 1, 148 Boundary Street  
West End, Brisbane  
QLD, Australia 4101



+61 7 3844 1182



[hello@qlicksmart.com](mailto:hello@qlicksmart.com)  
[www.qlicksmart.com](http://www.qlicksmart.com)



Search "qlicksmart"  
and stay connected



Scan the QR  
Code to learn  
more about the  
BladeSINGLE



Distributed by: