

BladeNeedleSYSTEM

Blade Remover | Needle Counter

Dual Function Scalpel Blade Remover and Needle Counter



www.clicksmart.com

 **CLICKSMART®**
... and be safe

The forgotten sharps injury

Scalpel injuries are up to 6 times more likely to occur than needlestick injuries. They are particularly common in surgical settings. Regrettably, healthcare organisations and personnel have underestimated the likelihood of these injuries.

Scalpel cuts can result in infections, blood-borne diseases and psychological distress. Serious injuries could require microsurgery to repair a digital artery, nerve and/or tendon.

Many such scalpel injuries can easily be prevented using Qlicksmart's **BladeNeedleSYSTEM** (BNS), the world's pioneering single-handed blade remover.

The world's pioneering single-handed blade remover



BNS is two safety devices in one - the world's only single-handed blade remover is combined with the standard needle counter to allow an accurate count and efficient disposal of both scalpel blades and suture needles.




BNS is simple to use with a quick "push, click, and pull" action to remove and contain used scalpel blades.



This easy-to-use solution provides a low cost alternative to safety scalpels, allowing surgeons to maintain their preferences for the standard metal handle.



Used together with a hands-free passing technique, the Qlicksmart BNS can prevent up to 5 times more injuries than a safety scalpel.

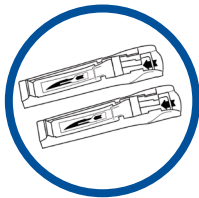

Extra blade remover cartridges are available for major surgical procedures.



It has already been embraced by healthcare workers around the world for its unique dual function, cost-saving and ease of inclusion into custom procedure packs.



The BNS contains the features of a numbered foam block for needles, a magnetic surface to hold odd-shaped sharps, and Qlicksmart's single-handed blade remover to remove used scalpel blades

Specifications

Product Code	QBNS2025	
GMDN Code	46236	
Blade Capacity	2 cartridges in the unit - each cartridge removes 1 (one) blade.	
Compatible Blades & Handles*	Handles: #3, 3L, 4, 4L, 5, 7, 9, baron, cylindrical. Blades: #10, 10a, 11, 11P, 12, 14, 15, 15a, 15c, 16, 19, 20, 21, 22, 22a, 23, 24, 25, 25a, 27, 36, 6, 9, E11, Sabre D/15.	
Incompatible Blades & Handles	Beaver handle, Disposable handles	
Size (per unit)	Closed: Length 11.7cm Width 6.6cm Height 3.9cm	Open**: Length 11.7cm Width 12.7cm Height 8.7cm
Weight (per unit)	51gr	

* Most brands will work with Qlicksmart BladeNeedleSYSTEM.

** Open box with cartridge standing in the holder


Material	Needle counter box: High Impact Polystyrene Cartridges: Polypropylene (PP)
Latex	No
DEHP	No
Sterile	Yes
Sterilization Method	Gamma radiation
Re-usable	No, each cartridge removes 1 (one) blade before disposal.
Disposal method	After the blade and needle counts have been performed, the needle counter and cartridges are disposed of separately as medical wastes in a sharps container
Regulatory approvals	Independently tested by a NATA accredited laboratory for compliance with AS 3825:2020 Standard (procedures and devices for the removal, containment and disposal of scalpel blades from scalpel handles). Australia - Listed under TGA ARTG No: 208058 USA - FDA listed Europe - Class I Sterile medical device under Council Directive 93/42/EEC and CE marked

BNSB160511A5


How to Use BladeNeedleSYSTEM

PREPARE USING TWO HANDS:

1. Remove the sterile packaging and peel the double-sided adhesive tape. Open the BladeNeedleSYSTEM and secure the device to a surface.

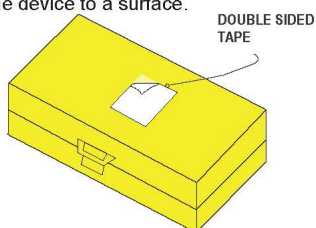


2. To remove a scalpel blade, place a cartridge into the round holder.

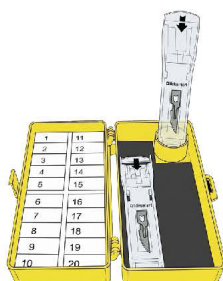


READY FOR USE

BLADE REMOVAL SINGLE HANDED:

3. Using one hand, slide the tip of the scalpel into the cartridge.

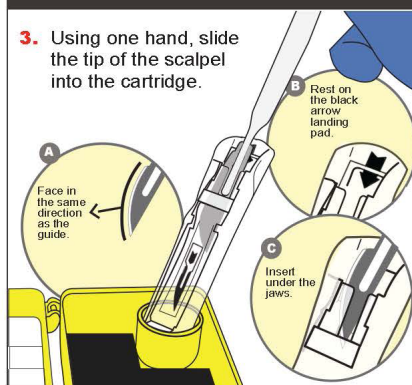


4. Tilt up to 90°, then push the scalpel down until you hear a click.

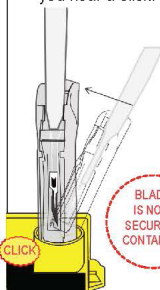


STERILE
Dispose of as medical waste

5. Remove the cartridge from the round holder.

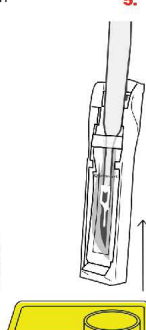


SAFE TO USE TWO HANDS

6. Using two hands, pull out the scalpel handle. The blade is now ready for counting.

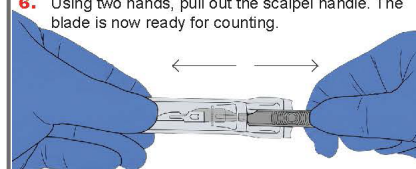


NEEDLE COUNTER

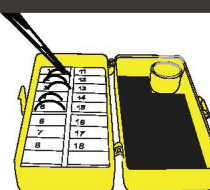

7. Insert the used needles into the foam as required. Larger needles can be placed on the flat magnetic surface.

DISPOSAL

8. Dispose of the needle counter and the cartridges separately into a sharps bin.

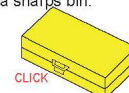

Scan the QR Code to see the BladeNeedleSYSTEM in action.
<https://www.youtube.com/watch?v=2J49LRQ0cw4>

CE EC REP Donaweg Lifescience
Piazza Albania, 10
00165 Rome, Italy

Other Qlicksmart Products

BladeFLASK EVO
Scalpel Blade Remover




BladeFLASK Scalpel Blade Remover



SnapIT and SnapIT Lite Ampoule Openers



BladeCASSETTE Sterile
Scalpel Blade Remover

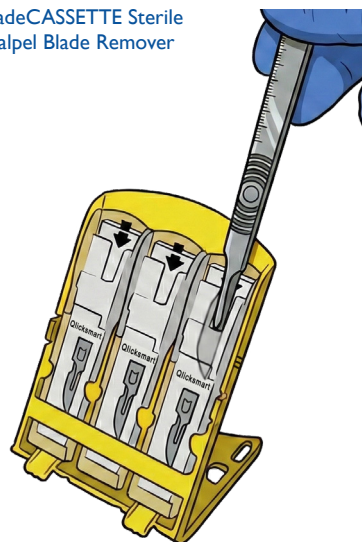

CheckCLIP Medication Identification



About Qlicksmart

Qlicksmart Pty Ltd is committed to making healthcare professional's lives easier and safer, so that they can focus on patient care. Established in 1999 by two Australian physicians to combat scalpel cuts in hospitals, our award-winning safety devices are now being used in over 50 countries worldwide. Qlicksmart works with innovators from the healthcare sector to deliver solutions which target the overlooked concerns of nurses, surgeons, and healthcare facilities. To learn more about Qlicksmart's safety devices, please visit www.qlicksmart.com or ask your distributor for a demonstration today.

Qlicksmart Pty Ltd.



Level 1, 148 Boundary Street
West End, Brisbane
QLD, Australia 4101



+61 7 3844 1182



hello@qlicksmart.com
www.qlicksmart.com



Search "qlicksmart"
and stay connected

Scan the QR Code
to learn more
about the BNS.



Distributed by: